# **C711 DM Specifications**





OKI is a leader in printing innovation, providing organisations with High Definition colour printing devices based on digital LED technology, and now our sophisticated range of printers integrate seamlessly with your clinical systems using DICOM

# A first in medical imaging and printing

This is a first in digital printer technology, combining the cost effectiveness and high quality output of an LED printer with embedded DICOM software, allowing you to print directly from medical equipment without the use of conversion software or external print servers. You no longer need to settle for expensive printing on specialist media and devices.

With the C711 DM you can now print your medical imaging cost effectively on standard media formats in vibrant, HD colour and in sizes up to A4. From x-rays (eg dental, veterinary, hospital, clinic) to nuclear medicine, scans (eg MRI, CT) and ultrasounds, these images can now be easily printed for your own non-diagnostic use or to provide a copy for your patients and clients to take away with them.

### Reduce your costs and improve your printing

The C711 DM is not only perfect for your medical imaging printing, it can also be used for your standard office printing, from patient letters and forms to medical reports, allowing you to choose one machine that answers all the needs of your organisation, cutting down on the number of devices and consumables used.

The C711 DM is based on our existing, award winning printers, offering the same reliability, ease of use, low total cost of ownership, media flexibility and high quality output, with the additional benefit of embedded DICOM technology. Through system integration, the use of energy efficient equipment, standard printer consumables and a three year on-site warranty, the C711 DM offers considerable savings for your day-to-day printing needs.

# See it clearer with OKI

Talking directly with your medical imaging equipment, the C711 DM presents many opportunities for printing high quality, non-diagnostic images in HD mono and colour on a wide range of media. The significantly reduced costs of printing using the C711 DM, compared with traditional DICOM printing technologies, allows medical establishments to change their approach to printing.

Imagine the possibilities; print MRI scans in colour, offer mothers-to-be A4 ultrasound scans or keep sharp and clear non-diagnostic images in patients' records, and all at a very low cost. In fact, the same cost of printing as standard office documents.

## Benefits at a Glance

- Fully integrated DICOM print server
- Up to 3 simultaneous imaging equipment connections
- Recommended usage Nuclear medicine, ultrasound, MRI/CT scans, x-rays
- High quality non-diagnostic mono and colour printing of medical images
- No additional external hardware or software required
- Handles media from A6 to A4 wide 1200mm banners, from 64 to 250gsm
- Three year on-site warranty (product to be registered within 30 days of purchase)



#### C711 DM - Colour and Mono DICOM Embedded Printer

Imaging Technology	Overview		
Network   10/100 Base TX (R/45) interface	Description		
Print Format - Paper weight Dimensions (HxWxD) 389 x 435 x 546mm  Weight 28kg  DICOM SERVER FEATURES  Capability DICOM 3.0 Print SCP  Maximum number of AET 25 Max # calling modalities No limitations 3 simultaneous Integrated web server  Log Extended log capability in case of troubleshooting identity, reverse, lin-od Specific color and mono rendering optimized for medical images Capability to add a graphical header on each printout AET DEPENDENT FEATURES  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Enter text that will print on top of page or select Logo Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove contrast on the printouts  Hi Quality Glossy Black Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Prompts C711 to use or not adjustable Sharpen filter to print Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  3rd paper tray 44274502  3rd paper tray 44274502	Imaging Technology		
Print Format - Paper weight Dimensions (HxWxD) 389 x 435 x 546mm Weight 28kg  DICOM SERVER FEATURES  Capability DICOM 3.0 Print SCP Maximum number of AET 45 Max # calling modalities No limitations Maximum connections Joicom Server interface Log Extended log capability in case of troubleshooting identity, reverse, lin-od Specific color and mono rendering optimized for medical images Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET) Printer Tray Use preferred paper for your medical prints Copy Send one, print what you need Border Density Value from Black, White or Client Empty image Density Value from Black, White or Client Use a graphical header instead of text for enhanced communication on the printouts Header Footer Edit the text on page bottom Contrast Add or remove contrast on the printouts Hi Quality Glossy Black Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Prompts C711 to use or not adjustable Sharpen filter to print Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 3rd paper tray 44274502	Network	10/100 Base TX (RJ45) interface	
Print Format - Paper weight Dimensions (HxWxD) Dimensions (HxWxD) DICOM Server Features  Capability DICOM 3.0 Print SCP Maximum number of AET Max # calling modalities Maximum connections Dicom Server interface Log Extended log capability in case of troubleshooting identity, reverse, lin-od Specific color and mono rendering optimized for medical images Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET) Printer Tray Use preferred paper for your medical prints Copy Send one, print what you need Border Density Value from Black, White or Client Empty image Density Value from Black, White or Client Use a graphical header instead of text for enhanced communication on the printouts Header Enter text that will print on top of page or select Logo Edit the text on page bottom Contrast Add or remove contrast on the printouts Hi Quality Glossy Black Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. Will lighten, values higher than 1.00 will darken the image Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Force image polarity inversion if needed  Accessories (Order Numbers) 2nd paper tray 44274502 3rd paper tray 44274502 3rd paper tray 44274502 3rd paper tray 44274502 3rd paper tray 44274502	<b>6</b>		
Dimensions (HxWxD)  Weight  Weight  28kg  DICOM Server Features  Capability  DICOM 3.0 Print SCP  Maximum number of AET  25  Max # calling modalities  Maximum connections  Joicom Server interface  Log  Extended log capability in case of troubleshooting  LUT support  identity, reverse, lin-od  Specific color and mono rendering optimized for medical images  Graphical Header  Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name  Print SCP Application Entity Title (AET)  Printer Tray  Use preferred paper for your medical prints  Copy  Send one, print what you need  Border Density  Value from Black, White or Client  Empty image Density  Value from Black, White or Client  Use a graphical header instead of text for enhanced communication on the printouts  Header  Enter text that will print on top of page or select Logo  Footer  Edit the text on page bottom  Contrast  Add or remove contrast on the printouts  Brightness  Add or remove contrast on the printouts  Hi Quality Glossy Black  Print your greyscale images using best composite rendering  Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Fromet Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray  44274502  3rd paper tray  44274502  Duplex  Duplex		A. (1. 25)	
DICOM SERVER FEATURES  Capability DICOM 3.0 Print SCP  Maximum number of AET 25  Max # calling modalities No limitations  Maximum connections 3 simultaneous  Dicom Server interface Integrated web server  Log Extended log capability in case of troubleshooting  LUT support identity, reverse, lin-od  Image generation Specific color and mono rendering optimized for medical images  Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Buplex 128kg  DICOM 3.0 Print SCP Applications  An image of the print SCP Application Entity Title (AET)  Dicom 3 print your greyscale images using best composite rendering for the print on			
Capability DICOM 3.0 Print SCP  Maximum number of AET 25  Max # calling modalities No limitations  Maximum connections 3 simultaneous  Dicom Server interface Integrated web server  Log Extended log capability in case of troubleshooting  LUT support identity, reverse, lin-od  Image generation Specific color and mono rendering optimized for medical images  Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601			
Capability         DICOM 3.0 Print SCP           Maximum number of AET         25           Max # calling modalities         No limitations           Maximum connections         3 simultaneous           Dicom Server interface         Integrated web server           Log         Extended log capability in case of troubleshooting           Identity, reverse, lin-od         Specific color and mono rendering optimized for medical images           Graphical Header         Capability to add a graphical header on each printout           AET Name         Print SCP Application Entity Title (AET)           Printer Tray         Use preferred paper for your medical prints           Copy         Send one, print what you need           Border Density         Value from Black, White or Client           Empty image Density         Value from Black, White or Client           Replace Header by Logo         Use a graphical header instead of text for enhanced communication on the printouts           Header         Enter text that will print on top of page or select Logo           Edit the text on page bottom         Add or remove contrast on the printouts           Add or remove brightness on the printouts         Add or remove brightness on the printouts           Fint your greyscale images using best composite rendering         Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values highe	weight	28Kg	
Maximum number of AET         25           Max # calling modalities         No limitations           Maximum connections         3 simultaneous           Dicom Server interface         Integrated web server           Log         Extended log capability in case of troubleshooting           LUT support         identity, reverse, lin-od           Image generation         Specific color and mono rendering optimized for medical images           Graphical Header         Capability to add a graphical header on each printout           AET Name         Print SCP Application Entity Title (AET)           Printer Tray         Use preferred paper for your medical prints           Copy         Send one, print what you need           Border Density         Value from Black, White or Client           Empty image Density         Value from Black, White or Client           Replace Header by Logo         Use a graphical header instead of text for enhanced communication on the printouts           Header         Enter text that will print on top of page or select Logo           Footer         Edit the text on page bottom           Add or remove contrast on the printouts           Hi Quality Glossy Black         Print your greyscale images using best composite rendering <th col<="" th=""><th>DICOM SERVER FEATURES</th><th></th></th>	<th>DICOM SERVER FEATURES</th> <th></th>	DICOM SERVER FEATURES	
Max # calling modalities         No limitations           Maximum connections         3 simultaneous           Dicom Server interface         Integrated web server           Log         Extended log capability in case of troubleshooting identity, reverse, lin-od           Specific color and mono rendering optimized for medical images           Graphical Header           Capability to add a graphical header on each printout           AET Name         Print SCP Application Entity Title (AET)           Printer Tray         Use preferred paper for your medical prints           Copy         Send one, print what you need           Border Density         Value from Black, White or Client           Empty image Density         Value from Black, White or Client           Use a graphical header instead of text for enhanced communication on the printouts           Replace Header by Logo         Use a graphical header instead of text for enhanced communication on the printouts           Header         Enter text that will print on top of page or select Logo           Footer           Add or remove contrast on the printouts           Hi Quality Glossy Black         Print your greyscale images using best composite rendering           Greyscale, Cyan, Magenta and	Capability	DICOM 3.0 Print SCP	
Maximum connections Dicom Server interface Log Extended log capability in case of troubleshooting LUT support identity, reverse, lin-od Specific color and mono rendering optimized for medical images Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES AET Name Print SCP Application Entity Title (AET) Printer Tray Use preferred paper for your medical prints Copy Send one, print what you need Border Density Value from Black, White or Client Empty image Density Value from Black, White or Client Replace Header by Logo Footer Edit the text that will print on top of page or select Logo Footer Edit the text on page bottom Contrast Add or remove contrast on the printouts Brightness Add or remove brightness on the printouts Hi Quality Glossy Black Print your greyscale images using best composite rendering Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 Juplex 01272601	Maximum number of AET	25	
Dicom Server interface Log Extended log capability in case of troubleshooting identity, reverse, lin-od Specific color and mono rendering optimized for medical images Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET) Printer Tray Use preferred paper for your medical prints Copy Send one, print what you need Border Density Value from Black, White or Client Empty image Density Value from Black, White or Client Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts Header Enter text that will print on top of page or select Logo Footer Edit the text on page bottom Contrast Add or remove contrast on the printouts Brightness Add or remove brightness on the printouts Hi Quality Glossy Black Print your greyscale images using best composite rendering Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 3rd paper tray 44274502 Duplex 01272601	Max # calling modalities	No limitations	
Log Extended log capability in case of troubleshooting  LUT support identity, reverse, lin-od  Specific color and mono rendering optimized for medical images  Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Maximum connections	3 simultaneous	
LUT support identity, reverse, lin-od  Image generation  Specific color and mono rendering optimized for medical images  Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Dicom Server interface	Integrated web server	
Image generation  Graphical Header  Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Log	Extended log capability in case of troubleshooting	
Graphical Header Capability to add a graphical header on each printout  AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	LUT support	identity, reverse, lin-od	
AET DEPENDENT FEATURES  AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print  Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Image generation	, ·	
AET Name Print SCP Application Entity Title (AET)  Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need  Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  Buplex 01272601	Graphical Header	Capability to add a graphical header on each printout	
Printer Tray Use preferred paper for your medical prints  Copy Send one, print what you need Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client Use a graphical header instead of text for enhanced communication on the printouts  Header Enter text that will print on top of page or select Logo Footer Edit the text on page bottom Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 3rd paper tray 44274502 Duplex 01272601	AET DEPENDENT FEATURES		
Copy Send one, print what you need Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  Duplex 01272601	AET Name	Print SCP Application Entity Title (AET)	
Border Density Value from Black, White or Client  Empty image Density Value from Black, White or Client  Replace Header by Logo  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  Duplex 01272601	Printer Tray	Use preferred paper for your medical prints	
Empty image Density  Replace Header by Logo  Header  Enter text that will print on top of page or select Logo  Footer  Contrast  Add or remove contrast on the printouts  Brightness  Add or remove brightness on the printouts  Hi Quality Glossy Black  Greyscale, Cyan, Magenta  and Yellow Gamma  Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing  Prompts C711 to use or not adjustable Sharpen filter to print  Polarity  Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray  44274502  Duplex  Duplex  Use a graphical header instead of text for enhanced communication on the printouts  Header  Enter text that will print on top of page or select Logo  Footer text that will print on top of page or select Logo  Footer few text that will	Сору	Send one, print what you need	
Replace Header by Logo  Header Enter text that will print on top of page or select Logo  Footer Edit the text on page bottom Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering Greyscale, Cyan, Magenta and Yellow Gamma Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 3rd paper tray 01272601	Border Density	Value from Black, White or Client	
Replace Reader by Logo communication on the printouts  Header Enter text that will print on top of page or select Logo Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502  Duplex 01272601	Empty image Density	Value from Black, White or Client	
Footer Edit the text on page bottom  Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print  Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Replace Header by Logo		
Contrast Add or remove contrast on the printouts  Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta and Yellow Gamma Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print  Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Header	Enter text that will print on top of page or select Logo	
Brightness Add or remove brightness on the printouts  Hi Quality Glossy Black Print your greyscale images using best composite rendering  Greyscale, Cyan, Magenta Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. will lighten, values higher than 1.00 will darken the image Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  3rd paper tray 44274502  Duplex 01272601	Footer	Edit the text on page bottom	
Hi Quality Glossy Black Greyscale, Cyan, Magenta and Yellow Gamma Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 Duplex Duplex Print your greyscale images using best composite rendering Range from 0.01 to 2.90 by step of 0.01. Values lower than 1. 00 will darken the image viil lighten, values higher than 1.00 will darken the image Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS) 2nd paper tray 44274502 Duplex 01272601	Contrast	Add or remove contrast on the printouts	
Greyscale, Cyan, Magenta and Yellow Gamma  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  Duplex 01272601	Brightness	Add or remove brightness on the printouts	
and Yellow Gamma will lighten, values higher than 1.00 will darken the image  Smoothing Prompts C711 to use or not adjustable Sharpen filter to print  Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray 44274502  Buplex 01272601	Hi Quality Glossy Black	Print your greyscale images using best composite rendering	
Polarity Force image polarity inversion if needed  ACCESSORIES (ORDER NUMBERS)  2nd paper tray  44274502  3rd paper tray  44274502  Duplex  01272601	, , ,	Range from 0.01 to 2.90 by step of 0.01. Values lower than 1.00 will lighten, values higher than 1.00 will darken the image	
Accessories (Order Numbers)  2nd paper tray	Smoothing	Prompts C711 to use or not adjustable Sharpen filter to print	
2nd paper tray     44274502       3rd paper tray     44274502       Duplex     01272601	Polarity	Force image polarity inversion if needed	
3rd paper tray 44274502  Duplex 01272601	Accessories (Order Numb	ERS)	
3rd paper tray 44274502  Duplex 01272601			
<b>Duplex</b> 01272601			
•			

#### CONSUMABLES (ORDER NUMBERS)

Toner cartridges* (11,500 pages)	Cyan: 44318607; Magenta: 44318606; Yellow: 44318605
Toner cartridges* (11,000 pages)	Black: 44318608
Image drum** (20,000 pages)	Cyan: 44318507; Magenta: 44318506; Yellow: 44318505; Black: 44318508
Transfer Belt (60,000 pages)	44341902
Fuser Unit (60,000 pages)	44289103

#### OFFICE USAGE FEATURES

C711 DM inherits all features of standard product	Please refer to C711 specsheet available on OKI websites
--	--

#### DICOM INTERFACE

On embedded web server  $\Big|$  Full configuration of the server features,  $\log$  viewing

AET Name	SCP AET Configuration OKI
Activate Max Dicom Log	
Use US format paper	
Printer Tray	AUTO ▼
Сору	CLIENT ▼
Border Density	CLIENT ▼
Empty Image Density	CLIENT ▼
Replace Header by Logo	
	Using Logo as Header reduce image printing area
Header	
Footer	DICOM Embedded Printer - Not for diagnostic use -
	Editing this original statement footer is full responsability of the user.
Contrast	0 •
Brightness	0 •
brightness	· ·
Hi Quality Glossy Black Mode	
Black gamma (0.01 to 2.90)	0.7
Cyan gamma (0.01 to 2.90)	0.85
Magenta gamma (0.01 to 2.90)	0.89
Yellow gamma (0.01 to 2.90)	0.82
Smoothing Filter	CLIENT
Smoothing Level	LOW
Polarity	CLIENT ▼



<sup>\*</sup>Toner: A4 pages yield compliant with ISO/IEC 19798. Printer shipped with toner for 4,000 pages. Medical printing strongly alter these values.

\*\*Drum: Average number of A4 pages printed.

Consumables Information: For the protection of your printer, and in order to ensure that you benefit from its full functionality, this model has been designed to operate only with genuine OKI toner cartridges. These can be identified by the OKI trademark. Any other toner cartridge may not operate at all, even if it is described as "compatible", and if it does work, your printer's performance and print quality may be degraded.



#### 3 Year On-site Warranty

Our devices are manufactured to the highest standards of quality and technology, which has been confirmed by independent tests. We are so convinced of the high quality of our products that we are offering you an extension of the standard warranty period to 3 years at no extra cost to you. Simply register your product within 30 days of purchase to be eligible for our unique all inclusive offer. For further information please visit www.oki.co.uk/warranty. Without registration, the standard pan-European 1 year warranty applies.

# OKI EUROPE LTD.

Blays House, Wick Road Egham, TW20 0HJ **United Kingdom** T+44(0) 20 8219 2190 F+44(0) 20 8219 2199 WWW.OKIEUROPE.COM

