

SuperCAST

High Resolution Wax for Direct Investment Casting

VERSION
3.0

Technical Datasheet

MSDS Available on Request



SuperCAST v3 offers enhanced part accuracy and faster build speeds compared to SuperCAST. Specifically designed for precision casting applications such as jewelry & dental.

Offering excellent strength, build speed and resolution, SuperCAST v3 is designed for direct-investment casting. Processing under a 12 hour burnout cycle, SuperCAST v3 burns beautifully leaving no ash or residue.

Suitable for:
Jewelry Casting
Dental Casting


Supplied in 500mL Material Packs containing:

- 1x 500mL Bottle of SuperCAST v3 Polymer
- 1x Build Tray

Note: This is the only on-going consumable for a Pico 3D Printer

Burnout Guide - Gypsum Bonded Investments

Note: These settings are to be used as a guide only and may require adjustment depending on casting conditions.

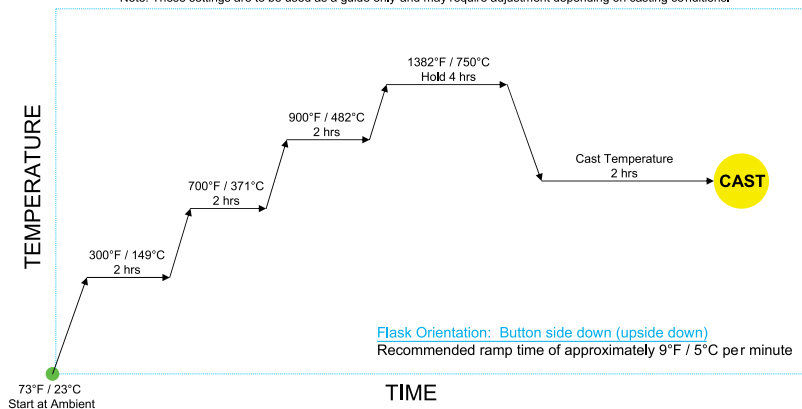

Flask Orientation: Button side down (upside down)
Recommended ramp time of approximately 9°F / 5°C per minute

Asiga recommends:
Investment: Kerr Satin Cast 20 from Kerr
Flask Type: Perforated Flask



ASIGA™
USA Toll Free - +1877 689 9998
www.asiga.com